Prenatal Listener®

OWNER'S MANUAL

KEEP INSTRUCTIONS FOR FUTURE USE.

©2009 Graco PD116361B 11/09

WARNING

Failure to follow these warnings and the assembly instructions could result in serious injury or death.

- **WARNING:** This product is not a toy.
- Product is intended for adult use only.
- **STRANGULATION HAZARD.** Keep the cords out of the reach of children.
- **DO NOT** immerse any part of the Prenatal Listener®, in water. Clean only with a dry cloth.
- **DO NOT** place unit near water and moisture. Do not use the Prenatal Listener® outdoors. Do not use the Prenatal Listener® near possible wet areas, such as a bathtub, shower, wash basin, sink, laundry tub, swimming pool, wet basement, etc.
- **Keep Prenatal Listener® away from heat sources**, such as fireplaces, radiators, stoves, and TVs or other appliances. Heat can damage the case or electrical parts.
- **DO NOT** use the Prenatal Listener® Unit if:
 - The cords or plugs have been damaged.
 - The unit has been exposed to liquids.
 - Unit has been dropped or is damaged.
- **DO NOT OPEN THE PRENATAL LISTENER®.** No user-serviceable parts inside. Risk of electrical shock, fire or death.

WARNING

FOR SAFE BATTERY

USE: Keep the batteries out of children's reach.

Any battery may leak battery acid if mixed with a different battery type, if inserted incorrectly (put in backwards) or if all batteries are not replaced or recharged at the same time. Do not mix old and new batteries. Do not mix alkaline, standard (carbon-zinc) or rechargeable (nickel-cadmium) batteries.

Any battery may leak battery acid or explode if disposed of in fire or an attempt is made to charge a battery not intended to be recharged. Never recharge a cell of one type in a recharger made for a cell of another type.

Discard leaky batteries immediately. Leaky batteries can cause skin burns or other personal injury. When discarding batteries, be sure to dispose of them in the proper manner, according to your state and local regulations.

Remove batteries before storing product for a prolonged period of time. Batteries left in the unit may leak and cause damage.

Recommended type batteries—alkaline disposables, size 9-volt (6LR61) not included. NEVER mix battery types. Change batteries when product ceases to operate satisfactorily.


Batteries should be disposed of properly by recycling.


Parts

Prenatal Listener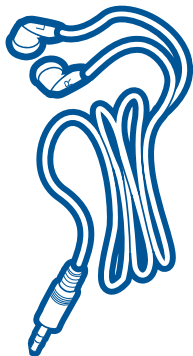

Earbuds (2x)

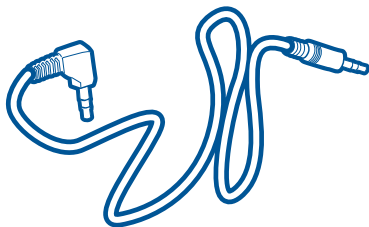Recording Cable

Listening to Unborn Baby ● ● ● ● ● ● ● ● ● ●

Introduction

Thank you for purchasing the Graco® Prenatal Listener®. Please read the following material carefully to maximize your enjoyment of this product.

It is important to understand that many factors affect what you will hear with the Prenatal Listener®. The position of your baby, your baby's weight, your weight, your stage of pregnancy, and where you place the Prenatal Listener® will all affect what you will hear.

For example, when listening for the fetal heartbeat, the sound will be loudest in the third trimester and if your baby is positioned with their back to your belly button. Your baby may naturally shift position, thereby creating the situation where you will hear your baby one moment but not the next.


However, once you've passed the second trimester, your baby will be growing very rapidly and you will hear the heartbeat and all the other sounds on a much more regular basis.

Don't worry! It's quite normal that there will be times you will not be able to hear your baby's sounds due to your baby's position or stage of development. This is especially true of the heartbeat which is highly dependent on your baby's position.



Please Read to Enjoy Our Product

- Please have patience. This product requires a little time but the results are wonderful. You should start using it at the beginning of your third trimester.
- What you will hear through the Prenatal Listener® will be the natural sounds of your baby. Your doctor uses Doppler ultrasound equipment which produces artificial noise by turning movement into sound. The Prenatal Listener® amplifies the actual sounds of your baby.
- The best time to listen is 3 to 4 hours after eating and when you are relaxed.
- The Prenatal Listener® must be used in a very quiet room.
- Placing the volume control at a low to moderate level will produce the best results and avoid static.
- You might not hear your baby's sounds immediately so don't be disappointed. The more you use the Prenatal Listener®, the better you will become at finding the sounds. Your baby's position will also greatly affect your success. The best position is with your baby's back to your belly button. Your success will also depend on how relaxed you are. You may have to try several times before you will hear your baby's sounds, particularly the heartbeat.
- Your baby's heartbeat does not sound like your own. A fetal heart beats from 120-180 times per minute. It will sound like a soft, rapidly beating drum.
- You will also hear kicks, hiccups, and nutrients passing through the placenta. Kicks sound like uneven thumps, hiccups like two rapid drum beats, and the nutrients like a whooshing sound.
- The Prenatal Listener® does not use any AC current, ultra sound, or radio waves.


Installing the Battery ●●●●●●●●●●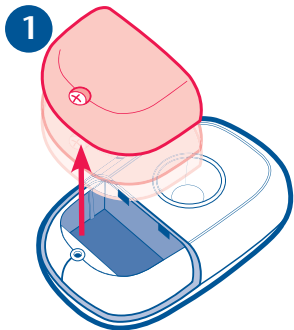


Loosen the screw and remove the battery cover and install a new 9-volt (6LR61) battery (not included). We recommend using a new battery as an old one will severely interfere with the performance of your Prenatal Listener®. Be sure to match positive (+) to positive (+) and negative (-) to negative (-).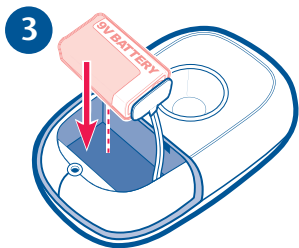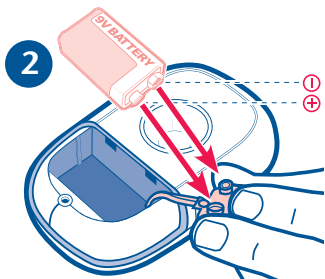

4

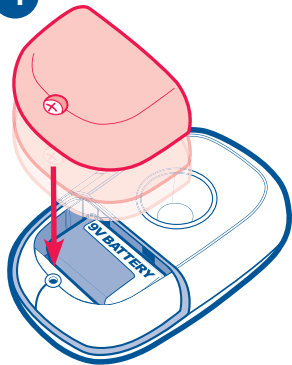
Replace the battery cover and tighten the screw.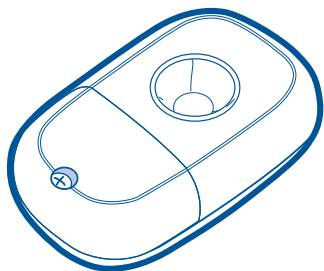


Listening to Your Unborn Baby ● ● ● ● ●

1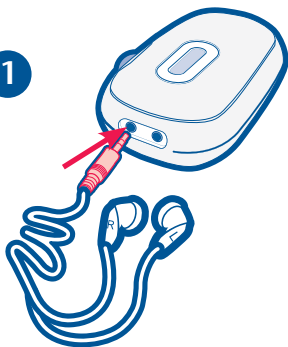


Insert the earbud plug into either jack on the unit and insert the earbuds into your ears.

2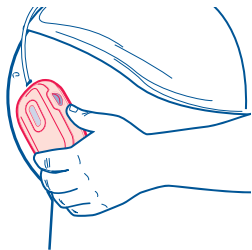


Hold the Listener to your stomach or lower back. Use the Prenatal Listener® on bare skin in a quiet room.

3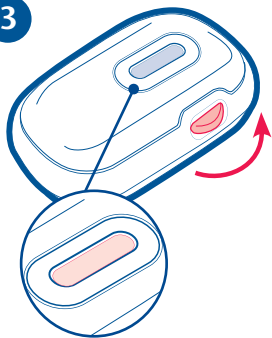


Turn the Prenatal Listener® on by turning the Power/Volume control wheel to the counterclockwise until you hear a click. The blue power indicator on the front will light up to indicate the unit is in the ON position.

4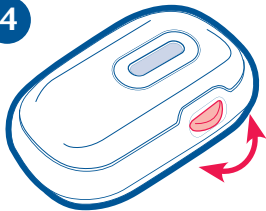


Adjust the volume to optimize the listening level and amplification. Begin with a low to moderate level to avoid static.

5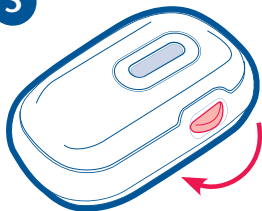


Before moving the Prenatal Listener®, turn the volume level down (turn clockwise) to the lowest position to avoid the sound of the cone moving against your skin.

6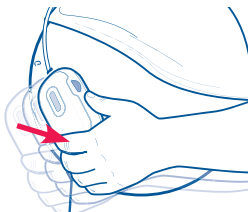

Start by placing the Prenatal Listener® two inches below and several inches to the right of your belly button. Gently move the unit one inch at a time to different places occasionally pausing until you find the best position for listening to your baby's sounds.


7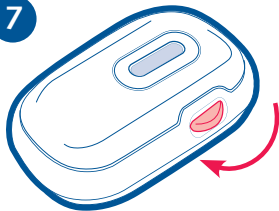

When you are finished, turn the Power/Volume control wheel all the way to the clockwise until you hear a click to turn the Prenatal Listener® off. The blue power indicator light will turn off.


To Record Unborn Baby ●●●●●●●●●●

1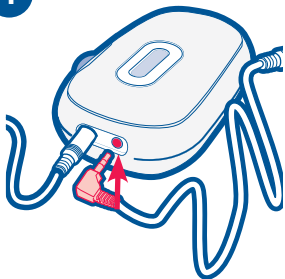


Plug the L-shaped end of the recording cable into either jack of the Prenatal Listener®.

2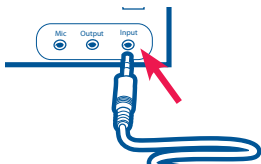

Plug the straight end into the line input of your recorder (some portable recorders do not provide this input). If your recorder does not have a line input, use the microphone input instead.

3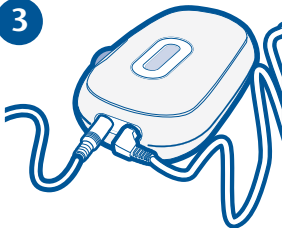

Set up the Prenatal Listener® for listening to your baby.

4

Adjust the recording level on your recorder if necessary.

5


Hold the Prenatal Listener® very still and begin recording. Remember, the sound of your heartbeat can have a very calming and soothing effect on your baby. You may want to record your own heartbeat before your baby is born. You will then be able to play back your recorded heartbeat to your baby after birth. We recommend a 30-minute recording.

NOTE: You will need to reduce the volume when you record your own heartbeat since it is much louder than that of your unborn baby's.

To Record and E-mail Using a Computer (if available)


If a sound recorder is available on your computer, you can use it to record your baby's sounds.

1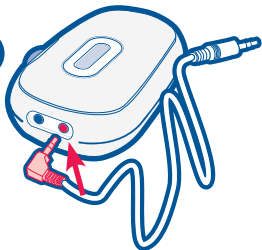

Plug the L-shaped end of the recording cable into either jack on the Prenatal Listener®.

2

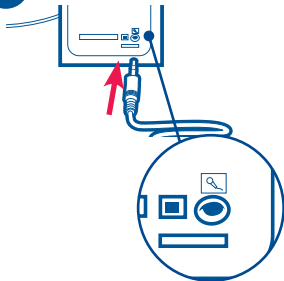Plug the end into the microphone jack on your computer. This is indicated by a picture of a microphone near the jack or the jack is color coded pink on the back of your computer.

3

Refer to your computer's User manual for detailed instructions on accessing and using your sound recording software.

Additional Product Information ● ● ● ●

Tips for Using the Prenatal Listener®

- The Prenatal Listener® has powerful amplification capabilities required to pick up the quiet sounds produced by your baby. To protect your ears and prevent feedback (squealing sounds), do not turn on the unit unless the listener is pressed against your body.
- There are many different sounds being produced by your baby and your body. Increasing the volume also increases these sounds. Keeping the volume at a moderate level usually produces the best results; therefore, adjust the volume carefully.
- Your baby's heartbeat is a very low volume sound and is very difficult to detect until the third trimester. Patience and very careful listening are required to locate the best position to hear the heartbeat. It is perfectly normal to hear it on some days and not others due to your baby's position and movements. It is also normal that you will hear it one moment and not the next. Don't worry! It is very natural for your baby to shift positions.

Troubleshooting ● ● ● ● ● ● ● ● ● ● ● ● ● ● ● ●

1. You can not hear your unborn baby with the Prenatal Listener®.

- a. It may be too early in your pregnancy to hear your baby. Wait until you are in your third trimester when the fetal heartbeat starts becoming louder.
- b. It is quite normal not to hear your baby due to their position. Try again later. The best time to listen is 3 to 4 hours after a meal and when you are relaxed. You may also want to try placing the Prenatal Listener® against your lower back as this may help minimize interference from your own body's sounds.
- c. Unit is not on. Turn unit on by turning the Power/Volume control wheel counterclockwise until you hear a click and the blue power indicator lights up.
- d. Volume control is turned down. Increase the volume by turning the Power/Volume control wheel counterclockwise.

e. The earbuds are not connected to the monitor. Make sure the earbuds are securely plugged into either jack on the monitor.

f. Battery is low. Blue power indicator light will dim. Insert a new 9-volt (6LR61) alkaline battery.

g. Battery is not inserted properly. Open the battery compartment and check to see that positive (+) is matched with positive (+) and negative (-) is matched with negative (-).

2. There is feedback (squealing noises) when you use your Prenatal Listener®.

a. The volume is too high. Turn the volume down by turning the wheel clockwise.

b. Lower the volume before moving the Prenatal Listener® against your skin.

3. You cannot record from the Prenatal Listener®.

a. The recording cable is not plugged in correctly. Make sure the end of the recording cable is securely plugged into either jack on the Prenatal Listener® and the straight end into the line input of your recorder.

b. Recorder cable is not in correct jack of recorder.

c. The volume control is turned down on your recorder. Increase the volume.

NOTE: Consult instruction manual for your recorder.

4. There is too much background noise and it makes a scratching sound when you move it.

Turn the volume down BEFORE you move the Prenatal Listener® from one spot to the next and don't turn it up again until you are no longer moving it against your stomach.

Care and Maintenance ● ● ● ● ● ● ● ● ● ●

- To clean the Prenatal Listener®, wipe with a soft dry or slightly damp cloth. Do not submerge in water or any other liquid.
- Store in a cool dry place.
- Keep out of the reach of children when not in use.
- Do not disassemble the Prenatal Listener®.
- Only use a 9-volt (6LR61) alkaline battery.
- During long periods of non-use, remove battery from the compartment.
- Please dispose of batteries properly and safely.
- Do not use or place open flames such as lighted candles on or near the unit.
- The Prenatal Listener® should be used at normal room temperature. Do not use in extreme heat or cold.

Disclaimer

THE GRACO® PRENATAL LISTENER® IS NOT A MEDICAL DEVICE.

The Graco® Prenatal Listener® is not a medical device and should not be used or relied upon in any way for diagnostic or other medical purposes. Any concerns about your pregnancy should be addressed with a medical practitioner. The Graco® Prenatal Listener® is not intended to be and should not be used as a substitute for regular prenatal care by an appropriate medical practitioner. You are encouraged to seek such prenatal care in connection with your pregnancy.

The Graco® Prenatal Listener® should be used only in accordance with the instructions provided herein, and the manufacturers and distributors of the Graco® Prenatal Listener® are not responsible for inappropriate use. The manufacturer expressly disclaims liability for incidental, consequential, or special damages of any nature.

Replacement Parts Warranty Information (USA)

To purchase parts or accessories or for warranty information in the United States, please contact us at the following:

Graco Children's Products
3 Glenlake Parkway
Atlanta, GA 30328

www.gracobaby.com
or
1-800-345-4109

Product Registration (USA)

To register your Graco product from within the U.S.A. visit us online at

www.gracobaby.com/productregistration.

We currently do not accept product registrations from those living outside the United States of America.

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