GUARDIAN

3-IN-1 ELONGATED STEEL COMMODOE

REF MDS89664EL

DO NOT install this product without first reading and understanding this instruction manual. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

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GENERAL WARNINGS

DO NOT install this product without first reading and understanding this instruction manual.

If you are unable to understand these warnings, cautions, and instructions, contact a healthcare professional or dealer before attempting to install this equipment.

- Prior to each use check that all joints and fittings on the commode are tight.
- Make sure all four legs are adjusted evenly and are free of any obstruction prior to use.
- Ensure all detent buttons are properly engaged in position prior to use.
- DO NOT exceed the maximum Weight Capacity listed on your commode (MDS89664EL: 400lb weight capacity).
- DO NOT twist or turn while seated on bedside toilet, this may cause frame to bend.
- DO NOT stand on commode.
- DO NOT use commode for any purpose that it was not intended to be used for.
- Check rubber tips for rips crack or wear. If these conditions exist, DO NOT use tips and replace IMMEDIATELY. (Rubber tips can be ordered REF: MDS89664T)
- The elongated commode is not to be used as a transfer bench, transfer device, or climbing device.

NOTE: Tips on this product contain dry natural rubber.
FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

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Assembly Instructions/Directives de montage/Instrucciones de montaje

You may wish to request a setup and operation demonstration with your local HME provider or pharmacy.

Component list:

1. Bedside Toilet
2. Backrest assembly
3. Seat and Lid assembly
4. Bucket, liner and Lid

Carefully remove the Bedside Toilet, Backrest, Seat and Bucket assemblies from the shipping containers.

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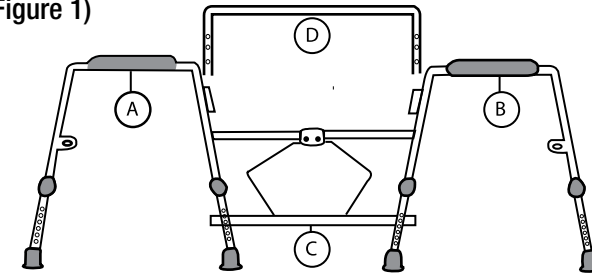
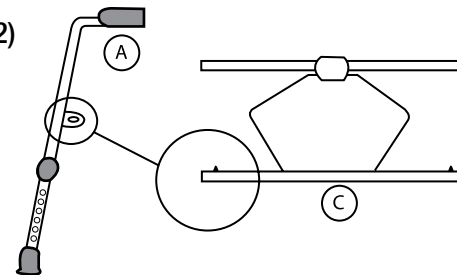
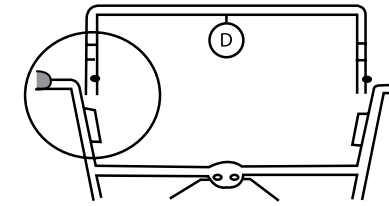
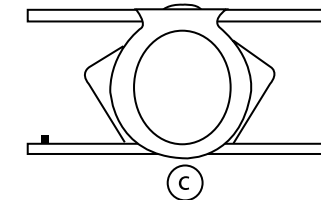
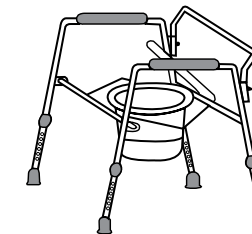
LIFETIME LIMITED WARRANTY

Your Medline brand product is warranted to be free of defects in material and workmanship for a lifetime for the original consumer. This device was built to precise standards and carefully inspected prior to shipment. This lifetime limited warranty is an expression of our confidence in the material and workmanship of our product and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse, negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories which are subject to normal wear and replacement. If you have any questions about your Medline product or this warranty, please call us at 1-800-MEDLINE. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com |
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2010 Medline industries, Inc. Guardian is a registered trademark of Medline Industries, Inc.

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